INSTRUCTIONS FOR USE

MUMU FFP2

MULTILANGUAGE: ENG, SLO, IT, DE, HR, HU

Table of Contents

<i>(ENGLISH) INSTRUCTIONS FOR USE</i>	2
<i>(SLOVENIAN) NAVODILA ZA UPORABO</i>	3
<i>(ITALIAN) ISTRUZIONI PER L'USO</i>	4
<i>(GERMAN) GEBRAUCHSANWEISUNG</i>	6
<i>(HRVATSKI) UPUTE ZA KORIŠTENJE</i>	8
<i>(HUNGARIAN) HASZNÁLATI ÚTMUTATÓ</i>	9

(ENGLISH) INSTRUCTIONS FOR USE

1. Open your mask by holding the laces. If necessary, you can loosen the laces a little by pulling on them. Thus, the first.
In use, you will prevent it from putting pressure on your ear.
2. Place the mask under your chin and cover your nose.
3. Pull the mask behind your ear, making sure it fits behind your ears.
4. Adjust the nose clip to minimize leakage.
5. Cover the mask with both hands and exhale. The mask should swell slightly. If air escapes from the sides of the mask, readjust the mask until it fits properly and make sure the nose clip is correctly positioned on your nose.

APPLICATION AND LIMITATIONS

These products meet the requirements of the EN 149:2001+A12009 standard.

CHECK BEFORE USE

- The user should choose the most suitable product according to the usage environment.
- It is obligatory to comply with the instructions for use specified by the manufacturer.
- Failure to comply with all instructions for use of this respiratory protective product and/or the entire exposure time

Failure to wear the entire product properly during the entire period may adversely affect the health of the user.

INSTRUCTIONS FOR USE

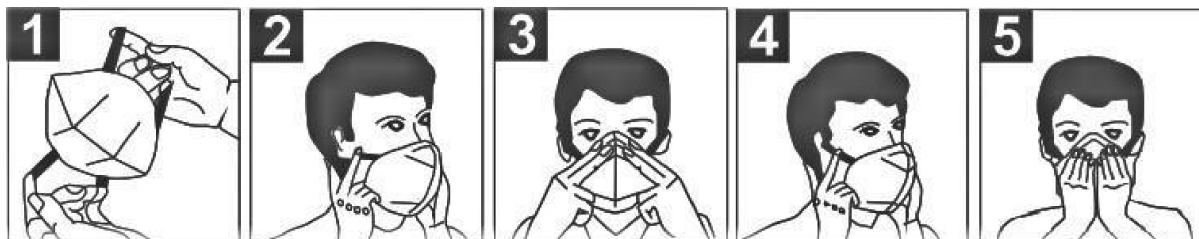
For your own safety, follow the instructions.

- The instructions for use of half masks that filter out particles should be read and followed.
- Half masks that filter out particles are made for the described use only.
- Wear your mask, which is suitable for the environment, as indicated in the mask usage figure.
- Check that the mask fits well on your face before entering the polluting environment or usage environment

CONVENIENCE CONTROL AND USE OF MASK AND FACE

- Check the fit of the mask to the face by adjusting it with both hands.
- If there is air leakage around your nose and non-ergonomic fit, make the necessary adjustment using the nose clip.
- Do not use in environments with an oxygen value lower than 19.5%.
- Do not use in narrow wells, caves, environments with unknown toxic pollutants.

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