# INSTRUCTIONS FOR USE

## Table of Contents

<i>ENGLISH: INSTRUCTIONS FOR USE</i>	<b>2</b>
<i>SLOVENSKO: NAVODILA ZA UPORABO</i>	<b>4</b>
<i>ITALIANO: ISTRUZIONI PER L'USO ITALIANO</i>	<b>6</b>
<i>DEUTSCHE: GEBRAUCHSANLEITUNG</i>	<b>8</b>
<i>HRVATSKI: UPUTSTVA ZA UPOTREBU</i>	<b>10</b>
<i>MAGYARORSZÁG: HASZNÁLATI ÚTMUTATÓ</i>	<b>12</b>

# ENGLISH: INSTRUCTIONS FOR USE

## MUMU PLUS EXTRA

Nitrile Powder Free Examination and Protective Gloves

***ENGLISH NITRILE POWDER-FREE EXAMINATION AND PROTECTIVE GLOVES.** Non-sterile, ambidextrous and for single use only. Manufactured from 100% synthetic nitrile polymer and does not contain natural latex. Store in cool, dry place and away from moisture. Do not use damaged packages. Gloves should be used according to their intended application.*

### STATEMENT AND CAUTION

- 1) This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals.
- 2) The penetration resistance and chemical resistance have **been assessed under laboratory conditions and relates only to the tested specimen** taken from the palm only (except in cases where the glove is equal to or over 400mm - where the cuff is tested also) and **relates only to the chemical tested**. It can be different if the chemical is used in a mixture.
- 3) It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation.
- 4) When used, protective gloves may provide less resistance to the dangerous chemical due to changes in the physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves.
- 5) Before usage, inspect the gloves for any defects or imperfections.
- 6) There are no possible allergen within the glove that are known to cause harm to the wearer.
- 7) After use wearers should visually check the gloves and remove any contamination from the outer surface before removing the gloves from the hand. Alternatively carefully peel the glove off the hand so that the contaminated glove outer does not touch your skin.
- 8) Gloves are worn on the hands of the healthcare and similar personnel to prevent contamination between healthcare personnel and the patient / user.

Gloves are medical device, they are resistant against bacteria and fungi and against viruses. They comply to MDR 2017/745 Class I and following standards: EN 420:2003+A1:2009, EU No. 10/2011 (this product is suitable for food handling, excluding acidic foodstuffs (tested for 2 hours at 40°C)), EN ISO 374-1:2016 Type B, EN ISO 374-1:2016, EN 374-4:2013, EN ISO 374-5:2016

Importer:

MY TICARET VE MEDİKAL ANONİM ŞİRKETİ  
Ömerli Mahallesi General Şükrü Koraltı Cad. No 33  
Arnavutköy / İstanbul / Türkiye  
T: + 90 212 438 20 64 / [info@mymedikal.com.tr](mailto:info@mymedikal.com.tr)

Manufacturer:

Top Glove Sdn Bhd  
Lot 4969, Jalan Teratai, Batu 6,  
Off Jalan Meru, 41050, Klang,  
Selangor D.E, Malaysia

Authorised Representative in European community

Top Glove Europe GmbH  
Bliersheimer Str. 80A,  
47229 Duisburg  
Germany

Distributor:

Nundinae group d.o.o.  
Vojkovo nabrežje 23  
6000 Koper

Notified body responsible for satisfying the requirement of PPE Regulation (EU) 2016/425 Cat III and module C2:

SATRA Technology Europe Limited.  
Bracetown Business Park Clonee Dublin 15 D15 YN2P.  
Republic of Ireland.  
(Notified Body Number (PPER): 2777)

Declaration of conformity available on: <https://nundinae-group.com>

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